A.P.B.®

EN All Purpose Boot
DE Verbandschuh
ES Bota multiusos
FR Chaussure polyvalente
IT Calzatura polivalente
NL Multifunctionele schoen
DA Universalsko
SV Universalsko
PL But wielofunkcyjny
SL Večnamenski čevelj
HR Univerzalna cipela
PT Bota multiúsos

Flat Sole

Rocker Sole

DARCO

EC REP

EN EU Authorised Representative
DE Bevollmächtigter in der EU
ES Representante autorizado en la UE
FR Représentant autorisé UE
IT Rappresentante UE autorizzato
NL Wettelijke vertegenwoordiger EU
DA EU-autoriseret repræsentant
SV Behörig representant i EU
PL Autoryzowany przedstawiciel w UE
SL Pooblaščen zastopnik EU
HR Ovlašteni zastupnik u EU
PT Mandatário na UE

UK REP

EN UK Responsible Person
DE Verantwortliche Person im Vereinigten
Königreich
ES Persona responsable en GB
FR Personne responsable Royaume-Uni
IT Persona responsabile nel Regno Unito
NL Verantwoordelijke rechtspersoon VK
DA Ansvarlig i Storbritannien
SV Ansvarig person för Storbritannien
PL Osoba odpowiedzialna w Wielkiej Brytanii
SL Odgovorna oseba v Združenem kraljestvu
HR Odgovorna osoba u UK
PT Responsável no Reino Unido

EN Instruction book symbol
DE Symbol Gebrauchsanweisung
ES Símbolo del manual de instrucciones
FR Symbole du mode d'emploi
IT Simbolo del libretto delle istruzioni
NL Symbool gebruiksaanwijzing
DA Se brochuren med anvisninger
SV Instruktionsboksymbol
PL Symbol instrukcji obsługi
SL Simbol knjige z navodili
HR Simbol priručnika s uputama
PT Símbolo de livro de instruções

EN Date of manufacture
DE Herstellungsdatum
ES Fecha de fabricación
FR Date de fabrication
IT Data di fabbricazione
NL Productiedatum
DA Fremstillingsdato
SV Tillverkningsdatum
PL Data produkcji
SL Datum izdelave
HR Datum proizvodnje
PT Data de fabrico

EN Certification mark declaring this product's compliance with EU legislation
DE Zertifizierungszeichen, das die Übereinstimmung dieses Produktes mit der EU-Gesetzgebung bestätigt
ES Marca de certificación que declara la conformidad de este producto con la legislación de la UE
FR Marque de certification attestant de la conformité de ce produit à la législation de l'UE
IT Marchio di certificazione che dichiara la conformità del prodotto con la legislazione UE
NL Certificeringskenmerk dat aangeeft dat dit product voldoet aan de wetgeving van de EU
DA Certificeringsmærke, der erklærer dette produkts overensstemmelse med EU-lovgivningen
SV Märket deklarerar att produkten uppfyller lagarna i EU
PL Znak certyfikacyjny deklarujący zgodność tego produktu z przepisami UE
SL Certifikacijska oznaka, ki potrjuje skladnost tega izdelka z zakonodajo EU
HR Certifikacijska oznaka koja označava sukladnost ovog proizvoda sa zakonodavstvom EU
PT Marca de certificação que declara a conformidade deste produto com a legislação da UE

EN Certification mark declaring this product's compliance with UK legislation
DE Zertifizierungszeichen, das die Übereinstimmung dieses Produktes mit der Gesetzgebung des Vereinigten Königreiches bestätigt
ES Marca de certificación que declara la conformidad de este producto con la legislación de GB
FR Marque de certification attestant de la conformité de ce produit à la législation de l'RU
IT Marchio di certificazione che dichiara la conformità del prodotto con la legislazione del Regno Unito
NL Certificeringskenmerk dat aangeeft dat dit product voldoet aan de wetgeving van de VK
DA Certificeringsmærke, der erklærer dette produkts overensstemmelse med UK-lovgivningen
SV Märket deklarerar att produkten uppfyller lagarna i Storbritannien
PL Znak certyfikacyjny deklarujący zgodność tego produktu z przepisami Wielkiej Brytanii
SL Certifikacijska oznaka, ki potrjuje skladnost tega izdelka z zakonodajo Združenega kraljestva
HR Certifikacijska oznaka koja označava sukladnost ovog proizvoda sa zakonodavstvom UK
PT Marca de certificação que declara a conformidade deste produto com a legislação do Reino Unido

EN Instructions for use

A.P.B.® | All Purpose Boot – Flat or Rocker sole

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a health-care professional. V-M Orthotics Ltd does not assume any responsibility. Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This device is not recommended for direct skin contact. You should use bandages, dressings or socks.

Do not drive a vehicle while you are wearing these shoes. Use extreme caution when walking with this device on uneven surfaces. Walk more slowly than usual. This device is intended for single-patient use only.

Purpose

The All Purpose Boot is to be used for any condition requiring support and protection following trauma or surgical procedures, casts and other bulky dressings and in combination with the PegAssist® Insole for treatment of wounds or ulcerations present under metatarsal heads and toes.

Indications / Contraindications

Indications

- > Used postoperatively for stability and mobilisation
- > Used with fibreglass cast bandages for the foot and ankle
- > In combination with the PegAssist® creates an interim solution for the diabetic foot in need of urgent care

Contraindications

- > Open fractures
- > Procedures using K-wires

Design features

- > The boot is fully closed and water-repellent
- > Low profile – no height compensation necessary
- > Replaceable insoles also available
- > Universal fit for right and left
- > The A.P.B.® flat sole version has a fixed dorsal flap with a left to right hook and loop closure.
- > The A.P.B.® rocker sole version uses a strapless fastening system which can be removed and rotated to provide a left or right overlap.

1. Open the boot (A).
2. Place foot into the boot.
3. Please ensure that the foot is securely positioned in the boot and that the heel area of the foot is as far back as possible in the boot.
4. Close boot with the hook and loop straps (A). The foot should be placed comfortably in the boot, not too loose and not too tight.

A.P.B.[®] rocker sole version:

Remove and rotate the fastening system to close the shoe from the inside to the outside.

Technical data

Sizes	XS (up to 4), S (4.5 – 6.5), M (7 – 8), L (8.5 – 10), XL (10.5 – 12.5) Can be worn on the left or right foot.
Colour	Black
Material	Outsole: TPR Insole: EVA Upper material: Polyester

Combination with other products

PegAssist[®] Insole

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling /cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Maintenance

The product does not require maintenance.

Cleaning instructions

Hand wash

- > If the shoe is slightly soiled, you can wipe it with a damp cloth or clean it with a soft brush.
- > Do not dry the shoe in a dryer or on a radiator. Leave it to dry at room temperature.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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DARCO International, Inc.
DARCO (Europe) GmbH