

Bodytonix Hinged Knee Support



The Bodytonix Hinged Knee Support provides excellent support and compression of the knee, which allows free flexion and extension at the knee and provides lateral and medial support via hinged side struts.



Indications for use:

- Mild to moderate ligament joint pain, sprain, strains of the knee ligaments.
- Ligament or meniscus injuries.

Product Features & Benefits:

- Designed to support, compress, stabilise and protect the knee to help improve injury recovery and prevent further injuries.
- Lightweight, breathable, skin friendly 3mm wrap around neoprene material with nylon lining, designed to keep your knee dry, sweat free and feeling comfortable even if you wear the knee brace for a long period of time.
- Medial and Lateral free-movement knee hinge with hyperextension stops
- Wrap-around neoprene sleeve to hold side hinges in correct position
- Open kneecap to keep the patella in the correct position, two splints for support to the sides of the joint and bi-directional compression straps.
- Removable hinges simulate the normal knee movement.
- Inbuilt patella pad surrounds kneecap to help improve patellar tracking.
- Removable felt cushion around the patella provides additional comfort: 2 adjustable straps provide a custom fit and supply additional compression.
- Comfortable and durable.
- Universal left or right.
- CE Mark, Class I medical device.



Instructions for use:

1. Sit down and step through the brace.
2. Bend your knee and align the hinge with the joint.
3. Secure the bottom strap first so it is snug but does not cut off circulation.
4. Secure the top strap around the thigh.
5. Secure the strap just below the knee, then the remaining straps.

Cautions:

- The use of this product should be supervised by a physician or health care specialist. In case of any abnormal sensation such as tingling, burning or numbness, pain, swelling or skin irritation, discontinue use and seek medical assistance.

Warranty:

- 12-month cover for defects in materials and workmanship in products purchased from Algeos only.

Care and Maintenance:

- Remove the liner and wash it by hand in cold water with a mild detergent. Do not use a washing machine or dryer. Place the liner flat to dry. Wash the plastic parts with a damp cloth and mild soap. Remove the plastic hinges and hand wash in cold water with mild soap solution.
- Reusable if not used on open wounds.

End of Use of the product:

Metal and Plastic-based materials may be recycled. Follow local environmental guidelines for waste disposal.

Code	Size	Thigh Circumference
BT0316	Small	33-37cm
BT0318	Medium	37-41cm
BT0320	Large	41-46cm
BT0322	X Large	46-53cm



A. Algeo Ltd
Sheridan House
Bridge Industrial Estate
Speke Hall Road
Liverpool
L24 9HB, UK

+44 (0)151 448 1228
+44 (0)151 448 1008
algeos.com
info@algeos.com