

Bodytonix Stabilising Knee Brace

Technical characteristics and usage advice

The Bodytonix Stabilising Knee Brace is ideally suited to provide knee support while reducing pressure to the kneecap.

Features & Benefits:

- Open kneecap to keep the patella in the correct position, two splints for support to the sides of the joint and bi-directional compression straps.
- Fits right or left leg.
- Proximal & distal straps.
- Adjustable with hook and loop fastening breathable uniform compression.

Technical Characteristics:

- Available in Small, Medium, Large and X-Large.
- Materials: neoprene sponge and woven nylon cover. Latex free.

Indications for Use:

- Mild to moderate injuries that require support while reducing pressure to the kneecap.
- Support for weak knees, mild ligament damage and pain relief from meniscus cartilage tear.
- For use during sport activities.

Cautions:

- Its use should be supervised by a physician or health care specialist, who should provide direction on frequency and duration of use.
- In case of any abnormal sensation such as tingling, burning or numbness, pain, swelling or skin irritation, discontinue use and seek medical assistance.

Storage:

Store at room temperature. Do not expose to direct sunlight.

Markings:

This product contains no additional markings other than the brand label.

Care Instructions:

Can be wiped clean with a damp cloth using a mild soap or disinfectant. Do not dry on a radiator.

Warranty:

12-month cover from purchase for defects in materials and workmanship.



Code	Size	Thigh Circumference
BT0424	Small	30cm - 34.5cm
BT0426	Medium	33cm - 37cm
BT0428	Large	35.5cm - 39.5cm
BT0430	X-Large	38cm - 42cm