

# Bodytonix Solid AFO Drop Foot Splint

## Technical characteristics and usage advice

The bracing effect of this AFO is a non-invasive low-cost way to hold the foot into normal position, not allowing it to drag on the ground.

For drop foot use due to peroneal nerve injury or sciatica nerve lesion and in painful conditions of the foot and ankle.

Clinician direction is fully advised when using this product - A custom device may be more suitable.

### Features & Benefits:

- Pre-fabricated polypropylene ankle-foot orthosis.
- Made of durable polypropylene material, designed specifically for right or left foot.
- Provide a dorsiflexion assist while walking.
- Low profile and easy to trim with scissors.
- Good 1st line device to monitor seriousness of condition.
- Recommended Sizing:

Size	Women's Foot Size (cm)	Men's Foot Size (cm)
Small	(8-15cm)	(13-19cm)
Medium	(15-23cm)	(19-27cm)
Large	(23-30cm)	(27-33cm)
XLarge	(30cm+)	(33cm+)

### Technical Characteristics:

- Available in Small, Medium, Large and X-Large.
- Latex free.

### Indications for Use:

- For use in drop foot due to peroneal nerve injury or sciatica nerve lesion.

### Cautions:

- Its use should be supervised by a physician or health care specialist, who should provide direction on frequency and duration of use.
- In case of any abnormal sensation such as tingling, burning or numbness, pain, swelling or skin irritation, discontinue use and seek medical assistance.

### Storage:

Store at room temperature. Do not expose to direct sunlight.

### Markings:

Brand label.

### Care Instructions:

Can be wiped clean with a damp cloth using a mild soap or disinfectant. Do not dry on a radiator.



Code	Size
BT1238	Left Small
BT1240	Left Medium
BT1242	Left Large
BT1244	Left X-Large
BT1246	Right Small
BT1248	Right Medium
BT1250	Right Large
BT1252	Right X-Large