

Bodytonix Solid AFO Drop Foot Splint

Technical characteristics and usage advice

The Swedish AFO provides Dorsiflexion Assistance and Lateral Stability.

This lightweight AFO is a moulded Polypropelene splint and can be used out-of-the-box or adapted with padding and additional straps as with a custom device.

Features & Benefits:

- Improve toe clearance during gait.
- Provides Dorsiflexion assistance.
- Customisable - Instep strap for equinus correction can be fitted.
- Proximal strap for support.
- Made of durable Polypropelene material.
- Designed specifically for right or left.
- Removable Proximal strap.
- Malleable - EVA lining can be added for a more bespoke, comfortable splint. Lightweight. Good rigidity.
- Sleek design - Low profile and open heel that does not affect how the shoe fits. High performance.
- Additional Distal strap can be fitted to AFO for increased support.
- Latex-free
- Recommended Sizing:

Size	Women's Foot Size (cm)	Men's Foot Size (cm)
Small	(8-15cm)	(13-19cm)
Medium	(15-23cm)	(19-27cm)
Large	(23-30cm)	(27-33cm)
XLarge	(30cm+)	(33cm+)

Indications for Use:

- Flacid foot drop foot due to peroneal nerve injury or sciatica nerve lesion.
- Foot drag due to hemiplegia.
- Neurological impairment.

Cautions:

- Its use should be supervised by a physician or health care specialist, who should provide direction on frequency and duration of use.
- In case of any abnormal sensation such as tingling, burning or numbness, pain, swelling or skin irritation, discontinue use and seek medical assistance.

Storage:

Store at room temperature. Do not expose to direct sunlight.

Care Instructions:

Can be wiped clean with a damp cloth using a mild soap or disinfectant. Do not dry on a radiator.

Warranty:

12-month cover from purchase for defects in materials and workmanship.



Code	Size
BT1258	Left Small
BT1260	Left Medium
BT1262	Left Large
BT1264	Left X-Large
BT1268	Right Small
BT1270	Right Medium
BT1272	Right Large
BT1274	Right X-Large