

Bodytonix

Bodytonix Dorsal Night Splint

Technical characteristics & usage advice

Anatomical fit, adjustable, lightweight, cost-effective.

The Bodytonix Dorsal Night Splint is a universal left or right fitting splint that can be used to treat multiple causes of heel pain and support correction whilst sleeping.

Indications for use:

Plantar fasciitis, Achilles tendinopathy, posterior tibial tendonitis, Achilles tendonitis, Metatarsalgia, Post-static pain, Hallux limitus, Sesamoiditis, Calcaneal Spur.

Features:

- Anatomical fit.
- Soft adjustable straps.
- Adjustable.
- Dorsal shell.
- 90° dorsiflexion.
- Soft lined foam interior.
- Soft foam padded interfaces.
- Washable.
- Breathable.
- Flexible and lightweight.

Benefits:

- Holds the foot in place without sacrificing comfort or mobility.
- The adjustable straps are quick and easy to use providing a secure and comfortable fit.
- Stretching and increasing flexibility in the calf reduces the risk of further strain and tearing of the plantar fascia.
- Soft lined foam interfaces provide maximum comfort during sleep.
- Durable.
- Cost-effective.
- Hygienic and washable components.
- Suitable for left or right foot.

Contraindications:

- You should not walk or attempt to place weight on the foot while using this splint without approval from your physician.
- Do not use outdoors, or on slippery or uneven surfaces.

Ordering Information:

Code	Size	Shoe
BT1378	Small/Medium	UK 5 - 9
BT1380	Large/X-Large	UK 9½ - 14



Care for your Night Splint:

- Liner and straps can be hand washed in cold water with mild soap. Air dry.
- Dorsal shell can be wiped with cold water and a mild soap or disinfectant. Air dry.

Materials:

- Soft pads: Polyester.
- Velcro® straps: Nylon.
- Dorsal shell: Plastic.