Movilo

EN Therapeutic Comfort Shoes
DE Therapeutische Komfortschuhe
ES Zapatos confort terapéuticos
FR Chaussures de confort thérapeutiques
IT Calzature comfort terapeutiche
NL Therapeutische comfort schoenen
DA Terapeutiske komfortsko
SV Ortopediska komfortskor
PL Terapeutyczne buty komfortowe
SL Terapevtski udobni čevlji
HR Terapeutske komforne cipele
PT Sapatos de conforto terapêuticos

Movilo Street

Movilo Fly

DARCO

EN	Instructions for use	2
DE	Gebrauchshinweis	5
ES	Instrucciones de uso	8
FR	Mode d'emploi	11
IT	Istruzioni per l'uso	14
NL	Gebruiksaanwijzing	17
DA	Brugsanvisning	20
SV	Bruksanvisning	23
PL	Instrukcja użycia	26
SL	Navodila za uporabo	29
HR	Napomena za uporabu	32
PT	Indicação de utilização	35
Sizes	38
Symbols	39

Movilo | Therapeutic Comfort Shoes

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your doctor, specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, they will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, instability, etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This product assists recovery and should only be used in accordance with the instructions for use. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This product is not recommended for direct skin contact. Use socks.

Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Be particularly cautious when walking in wet, slippery and snowy conditions.

Note: This product is allowed for single-patient use only. It does not need to be cleaned or reprocessed between uses.

If you feel unstable, check the shoe size and lacing.

Purpose

The therapeutic comfort shoe is used to redistribute pressure and wide-area pressure absorption. The shoe offers sufficient room in the toe area for pressure-sensitive and deformed toes.

Indications / Contraindications

Indications

- For the treatment of foot complaints:
- > Slight foot/toe deformities such as hallux valgus, hammer and claw toes
 - > Hallux rigidus
 - > Metatarsalgia
 - > Morton's neuroma
 - > Rheumatoid arthritis
 - > Polyarthritis, arthrosis, gout
 - > Fasciitis plantaris / heel spur
 - > Callus formation, panaritium prophylaxis
 - > Diabetes mellitus without PNP / PAOD (risk group 0 according to the German Diabetes Society)

Contraindications

- > No relief of the hip joint possible
- > Acute lesions and ulcerations
- > Florid DNOAP

EN Design features

The shoe provides stable heel support. Toes are protected by a high toe box. The anatomical arch cushion insole can be replaced with an individual orthopaedic insole. The 2 mm thick insole under the arch cushion insole allows the shoe to adapt to the foot volume. The fasteners are easy to set. The outsole has soft shock absorption with a light rearfoot rocker, a back-shifted ball rocker and a stable footprint. An additional stiffened sole is integrated into the outsole for off-loading of the forefoot.

Use

1. Wear suitable socks.
2. Check the correct shoe size. To do this, remove the insoles from the shoes and position the feet on the insoles.
3. Put weight on your feet.
4. The toes should have between 9 - 15 mm room to the front of the insoles.
5. Put the insoles back into the shoes.
6. Press the lace button on the quick-release fastener and slide it to the end of the laces.
7. Open the shoes as wide as possible.
8. Position your feet in the shoes. Use a shoe horn if necessary.
9. Press the lace button on the quick-release fastener and slide it back. The feet should be stable, but not be too tight.
10. Walk a few steps to check the fit.

The following features can be adjusted:

Too little room?

- > Remove the 2 mm insole under the arch cushion insole.

Use of an orthopaedic insole:

- > Remove the two insoles (6 mm arch cushion insole + 2 mm placeholder insole)
- > Insert the individual custom-made orthopaedic insole and check the fit.

Too little support in the shoe?

- > Remove the rubber laces and instead use the non-elastic laces supplied.

Too little heel support?

- > Also use the laces in the last eyelet.

Combination with other products

No information

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / care. With proper shoe care, a functionally reliable service life of 1 year is to be expected. After the usual duration of use, check the function at your specialist dealer. If used improperly (e.g. shoe care that deviates from the instructions), the usual wear and tear is increased. The statutory warranty, as described in our general terms and conditions and consumer information, always applies.

EN Technical data

Colour

Movilo Fly

Colours may vary depending on the collection.

Movilo Street

Colours may vary depending on the collection.

Material

Movilo Fly

Upper material: Polyester, elastane, TPU

Liner: Polyester

Outsole: Rubber, EVA

Stabilising element: Nylon

Insole: PU, nylon, EVA

Fastener: Nylon

Elastic laces: Rubber, polyester

Non-elastic laces: Cotton, polyester

Adhesive: Water-based PU adhesive

Movilo Street

Upper material: Polyester, nylon, PU

Liner: Polyester

Outsole: Rubber, EVA

Stabilising element: Nylon

Insole: PU, nylon, EVA

Fastener: Nylon

Elastic laces: Rubber, polyester

Non-elastic laces: Cotton, polyester

Adhesive: Water-based PU adhesive

Maintenance

The product does not require maintenance.

Care instructions

1. Remove coarse dirt from the outsole and upper material with a dry soft shoe brush.
2. Pull the laces out of the shoes and clean them by hand with mild soap.
3. Apply a mild soap solution to a soft brush or cloth. Clean the outsole thoroughly with it. Dry the soles with a soft cloth.
4. Remove the insoles and wash them by hand with mild soap. Make sure the insoles are completely dry before putting them back in the shoes.
5. The upper material can be cleaned with a damp cloth. Do not rub too hard to avoid damaging the materials.
6. Do not dry the shoes in a dryer or on a radiator. Leave it to dry at room temperature. Use shoe trees if necessary.

Note: Do not use any aggressive cleaning agents or bleach.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product in accordance with applicable local laws and regulations.

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A. Algeo Ltd
V-M Orthotics Ltd.
ALBO-Healthcare GmbH