Indications

- > Postoperative use
- > Trauma care
- > Stress fractures
- > Soft tissue injury

Contraindications

- > Open fractures
- > Poor circulation

Cleaning Instructions/Maintenance

- > When the shell or inner liner is slightly dirty, clean it with a damp cloth or a soft brush.
- > Allow the shell or inner liner to air dry only.

Instructions for use

1. Open the hook and loop straps and release the closures of the inner liner.
2. Spread open the liner and place the foot heel first into the walker.
3. Wrap the liner around the foot and secure with the closures.
4. With leg in walker and ankle positioned at 90°, begin securing hook and loop straps, starting at the toes and working towards the knee.

Additional Steps: Tall-Air and Short-Air versions ONLY

5. Inflate liner by pressing the pump. Caution: DO NOT OVERINFLATE.
6. To release air pressure, press the silver button located beneath the pump until air begins to escape.

Caution

Please always follow these instructions for use. In case of adverse effects (such as pain or swelling) discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO® International Inc. does not assume any responsibility for misuse of the product. It is advisable to wear dressings or socks with the walker. Do not wear it in direct contact with injured, irritated, damaged or broken skin. Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal. This device is for single-patient use only.

To the patient/user: This device has been designed and manufactured by DARCO® International Inc. to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. If you report this to the manufacturer, they will forward this immediately. When treatment is complete, dispose of this device according to local laws and ordinances.

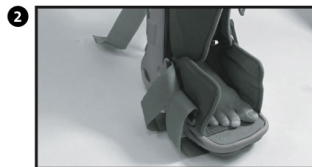
Manufactured for:
 DARCO® International, Inc.
 810 Memorial Blvd.
 Huntington, WV 25701, USA
 Phone 304.522.4883
 www.darcointernational.com

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