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Manufactured for:
 DARCO® International, Inc.
 810 Memorial Blvd.
 Huntington, WV 25701, USA
 Phone 304.522.4883
 www.darcointernational.com

V-M Orthotics Ltd.
 Member of hg DARCO Group
 Unit 25 | Halesworth Business Centre
 Norwich Road | Halesworth
 Suffolk IP19 8QJ | United Kingdom

Made in China

GS-IFU-23_Rev0123

DARCO®

GentleStep™

Instructions For Use

Indications

- > Patients diagnosed with diabetes
- > Peripheral neuropathy
- > Bony abnormalities of the foot

Contraindications

- > Open fractures

Cleaning Instructions/Maintenance

- > When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.
- > For heavily soiled shoes, machine wash on gentle cycle or handwash in mild detergent.
- > Allow the shoe to air dry only.

Instructions for use

The special fitting mark  located on the side of the sole assists in the proper fitting of the shoe. The ball of the foot or big-toe joint should be located at or just behind this marker. Correct fitting is essential to receiving maximum benefit from this professionally-designed shoe.

1. Open the shoe by means of the closures.
2. Please ensure that the foot is properly positioned in the shoe.
3. Secure the foot in the shoe with the closures.

Caution

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO International does not assume any responsibility. Do not wear the product with injured, irritated, damaged or broken skin. You should use bandages or socks, for example. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This aid is intended for single-patient use only.

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