TAS[®] Toe Alignment Splint

- EN Toe Alignment Splint
- DE Hallux Valgus Zehenschiene
- ES Férula para juanete
- FR Attelle pour hallux valgus
- IT Ortesi correttiva per alluce valgo
- NL Hallux valgus-teenspalk
- DA Hallux valgus tåskinne
- SV Tåskena för hallux valgus
- PL Szyna korekcyjna na paluch koślawy
- SL Opornica za izkrivljen nožni palec (hallux valgus)
- HR Odstojnik za nožne prste za Hallux Valgus
- PT Tala de dedo para hallux valgus

DARCO

algeos **ALGEOOS**
www.algeos.com.au

A. Algeo Ltd
Sheridan House
Bridge Industrial Estate
Speke Hall Road
Liverpool
L24 9HB, UK

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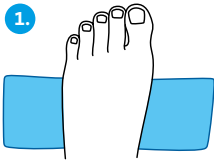
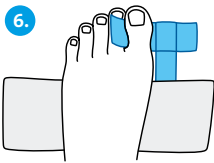
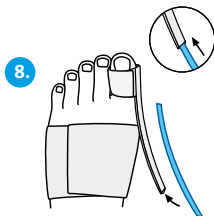
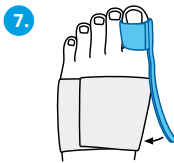
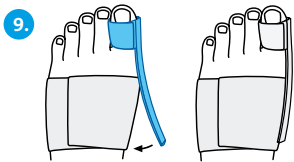
TAS® (Toe Alignment Splint) | Toe Alignment Splint

 **Important notes:** If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities. Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This device is not recommended for direct skin contact. You should use bandages, for example. While wearing this aid, ensure sufficient forefoot clearance and avoid all kinds of outside mechanical influences. This aid is allowed for single-patient use only. It does not need to be cleaned or reprocessed between uses.

Purpose

The bandage is for post-operative use to maintain the surgical result.

Indications / Contraindications

Indications

Post-operative use after:

- > Hallux valgus surgery
- > Tailor's bunion surgery

- > Hammer toe surgery

Contraindications

No contraindications are known.

Design features

The TAS® Toe Alignment Splint is a postoperative bandage for alignment that is used instead of taping. It enables positioning and alignment of toes in the correct position after surgery/correction, which has a decisive impact on the surgical result. A strap at the back of the foot reduces slippage of the midfoot bandage. The soft material makes it comfortable to wear.

Use

Hammer toes (fig. 1 – 3):

The elastic metatarsal band is designed so that it can be adjusted to the foot individually. Place the band around the foot as shown (fig. 1 – 2).

The heel strap prevents the bandage from slipping and is put on as shown in fig. 3.

Tailor's bunion (fig. 4 – 5):

The small toe loops are fixed to the metatarsal band according to need using the hook and loop strap (fig. 4 – 5).

Hallux valgus (fig. 6 – 9):

The big toe loop is fixed as shown in fig. 6 – 7. The stabilising element can be inserted into the big toe loop (fig. 8 – 9).

Combination with other products

No specifications

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling/cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Technical data

| | | | |
|------------------|--------------------------------------------------------|-----------------------|----------------------------------------------|
| Size: | Universal; Can be worn on the right and left | | |
| Colour: | White | | |
| Material: | Elastic strap: Polyester, polyamide, viscose, elastane | Hook strap: Polyamide | Label: Polyester satin |
| | Fleece strap: Polyamide | Velour: Polyamide | Stabilising element: Polyethylene, polyester |
| | Tag: Polyester satin | | Sewing thread: Polyester |

Maintenance

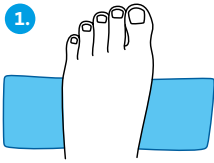
The product does not require maintenance.

Care instruction

- > Hand wash: Cold or lukewarm, max. 30°C
- > Do not bleach
- > Do not dry in dryer
- > Do not iron
- > Do not dry-clean

Disposal

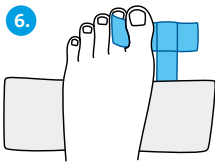
 Dispose of after use. Please dispose of the packaging in an environmentally manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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