

Softie™ Shoe



DARCO

Instructions for use



Softie™ Shoe

Postoperative Shoe

The DARCO Softie™ Shoe was designed specifically for use after soft tissue procedures or trauma to the foot.

1. Open the shoe by means of the two hook and loop straps (A).
2. Please ensure that the foot is securely positioned in the shoe.
3. This shoe can be worn on the left or right foot.
4. Close the shoe by means of the hook and loop straps.

Indications

- > For use after soft tissue procedures or trauma to the foot

Contraindication:

- > Open fractures

Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please inform your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. It is advisable to wear dressings or socks with the shoe. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This aid is intended for single-patient use only.

Technical data

Sizes:

3 Women sizes:

WS (up to 4), WM (4.5 – 5.5), WL (6 – 7.5)

4 Men sizes:

MS (up to 8), MM (8.5 – 10), ML (10.5 – 12), MXL (12.5 – 14)

Right and left fit

Colour: Black

Material:

Outsole: TPR

Insole: EVA

Upper material: Tri-laminated tricot-foam

Straps: Nylon

Maintenance: The product does not require maintenance.

Construction features:

The Tri-laminated upper part is made of light weight breathable tricot-foam. The Rubber sole is made of TPR with shock absorbing effect. The sole profile enables an easier rolling-off movement.

Cleaning instructions

1. When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.
2. Hand washing: the Softie™ Shoe can be hand washed in cold water with mild soap and may be air dried afterwards. Please do not use strong solvents for cleaning (alcohol etc.).
3. Do not tumble dry or dry using heat. Allow the shoe to air dry at moderate temperatures.



DARCO International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA
Toll-free 800 999 8866
Phone 304 522 4883
Fax 304 522 0037
www.darcointernational.com

EC REP

DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting | Germany
Tel. +49 88 07 9228-0
Fax +49 88 07 9228-22
info@darco-europe.com
www.darco-europe.com



ALGEOS

A. Algeo Ltd
Sheridan House
Bridge Industrial Estate
Speke Hall Road
Liverpool
L24 9HB, UK

- +44 (0)151 448 1228
- +44 (0)151 448 1008
- algeos.com
- info@algeos.com

Made in China



210331