

# Bodytonix

## Bodytonix Hybrid Night Splint

Technical characteristics & usage advice

**Adjustable, comfortable, lightweight, cost-effective.**

The Bodytonix Hybrid Night Splint is a uniquely-designed, universal left or right fitting splint that encompasses the benefits of the Bodytonix Dorsal Night Splint.

The Bodytonix Hybrid Night Splint can be used to treat multiple causes of heel pain and supports correction whilst sleeping. It provides an adjustable plantar fascia stretch whilst boasting maximum comfort.

### Indications for use:

Plantar fasciitis, Achilles tendinopathy, Posterior Tibial Tendonitis, Achilles Tendonitis, Metatarsalgia, Post-static pain, Hallux limitus, Sesamoiditis, Calcaneal Spur.

### Features:

- Anatomical fit.
- Adjustable tension strap.
- Adjustable flexion/stretch hinge.
- Textured silicone sole.
- Low profile splint.
- Soft lined foam interior.
- Thick, soft foam padded interfaces.
- Washable.
- Breathable.
- Flexible and lightweight.
- Hook and Loop fastenings.

### Benefits:

- Holds the foot in place without sacrificing comfort or mobility.
- Fully adjustable tension strap accommodates individual patient tolerances and improves compliance.
- Articulating hinge allows for a full range of adjustable settings throughout rehabilitation (90° +/- 15°).
- Soft lined foam interfaces provide maximum comfort during sleep.
- Foot grip on the soft interfacing surface to prevent slipping.
- Durable.
- Cost-effective.
- Hygienic and washable components.
- Suitable for left or right foot.

### Contraindications:

- You should not walk or attempt to place weight on the foot while using this splint without approval from your physician.
- Do not use outdoors, or on slippery or uneven surfaces.

### Ordering Information:

Code	Size	Shoe
NV3348	Small/Medium	UK 5 - 8½
NV3349	Large/X-Large	UK 9 - 13½



Foot grip on the interfacing surface to prevent slipping.

Position indicator.

### Care for your Night Splint:

- Liner and straps can be hand washed in cold water with mild soap. Air dry.
- Internal shell can be wiped with cold water and a mild soap. Air dry.

### Materials:

- Soft pads: Polyester.
- Velcro® straps: Nylon.
- Outer shell: Plastic.