EN Instructions for use

PegAssist® Insole
Off-loading Insole

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.
Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This aid is intended for single-patient use only.

Purpose

The PegAssist® Insole is an effective insole system for redistribution of plantar pressure.

Indications / Contraindications

Indications
For selective off-loading for:

- > Plantar lesions
- > Plantar ulcers
- > Pressure points

Contraindications
> Open fractures

Design features

Removal of single pegs enables customization – postoperatively or for acute wound care. A thin support board stabilises the off-loading area, and keeps the remaining pegs in place. The pressure is reduced by up to 50%.

Use

1. Remove the original sole from the shoe (fig. 1).
2. Insert in return the PegAssist® Insole with the smooth, continuous side facing upwards (pegs facing downwards) into the shoe (fig. 2).
3. Place a sterile film over the wound or pressure ulcer (fig. 3) and draw a circle on the film around the affected area with the provided marker pen (fig. 4).
4. The patient should now stand up on the insole in the shoe so that the marked area is transferred onto the blue upper layer of the insole (fig. 5).
5. Carefully remove the pegs beneath the marked area where the pressure should be relieved (fig. 6).
6. Apply the complete surface of the cardboard cover (included in the delivery) to the insole with the peg. The cardboard has an adhesive backing which fixes the pegs surrounding the off-loaded area and ensures that these do not shift (fig. 7).
7. Reinsert the adjusted PegAssist® Sole in the shoe (with the cardboard cover facing downwards) (fig. 8).
8. Fitting finished (fig. 9).

Note:

Signs of fatigue can occur in the cover layer when there is a large gap.

Combination with other products

- Insole with blue pegs for:
- > MedSurg, MedSurg Pro® – Healing Shoe
 - > Relief Dual® – Off-loading Shoe
 - > Relief Dual® Plus – Off-loading shoe with high ankle support
 - > Relief Insert® Walker – Lower Leg/Foot Orthosis

Insole with black pegs for:

- > OrthoWedge – Off-loading Shoe
- > OrthoWedge – Light Off-loading Shoe
- > HeelWedge® – Off-loading Shoe
- > Commodus Open® – Open Healing Shoe
- > SlimLine® – Cast Boot

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling/cleaning.

Technical data

- Sizes:**
- > Insole with blue pegs available in 7 sizes
 - > Insole with black pegs available in 5 sizes
 - > Can be worn on the right and left

Colour: Black, blue
Material: The 18 mm thick insole consists of 3 layers: A combination of EVA (blue or black), Plastazote® and a top layer of Poron®.

Maintenance

The product does not require maintenance.

Cleaning instructions

The soles can be cleaned with lukewarm water and soap or with conventional disinfectants.

Disposal

Dispose of after use Please dispose of the packaging in an environmentally friendly manner. The product can be disposed of with household waste.

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