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Body Armor® Stirrup Walker | Lower Leg Foot Orthosis

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO International Inc. does not assume any responsibility. Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This device is not recommended for direct skin contact. You should use bandages, dressings or socks.

Do not drive a vehicle while you are wearing this device. Use extreme caution when walking with this device on uneven surfaces. Walk more slowly than usual. This device is intended for single-patient use only.

Purpose

Lower leg foot orthosis for immobilization in a given position

Indications / Contraindications

Indications

- Lower leg-foot region for:
- > Postoperative use
- > Trauma care
- > Stress fractures
- > Soft tissue injury

Contraindications

- > Open fractures
- > Poor circulation

Technical data

Sizes

- > Version high and low
- > Universal fit for right and left

Size	EU	UK
XS	34 – 36	up to 4
S	37 – 39	4.5 – 6.5
M	40 – 42	7 – 8
L	43 – 45	8.5 – 11
XL	46 – 48	12 – 13.5

Material

Outsole: EVA
Insole: TPR
Lining: Polyester
Hook and loop straps: Nylon
Splints: Polypropylene
Pump: TPU

Combination with other products

No specifications

Design features

Shock-absorbing rocker outsole with lateral stabilization elements for immobilization. Closing system with variable hook and loop fasteners. Optional inflatable liner in both high and low versions for individual adjustment. The lower leg foot orthosis is available in five sizes with a universal fit for the right and left.

Use

- Open the hook and loop straps and release the closures of the inner liner.
- Spread open the liner and place the foot heel first into the walker.
- Wrap the liner around the foot and secure with the closures.
- With leg in walker and ankle positioned at 90°, begin securing hook and loop straps, starting at the toes and working towards the knee.

Additional Steps: High-Air and Low-Air versions ONLY

- Inflate liner by pressing the pump.
- To release air pressure, press the silver button located beneath the pump until air begins to escape.

Caution

Do not overinflate air versions.

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Maintenance

The product does not require maintenance.

Cleaning instructions

- > When the shell or inner liner is slightly dirty, clean it with a damp cloth or a soft brush.
- > Allow the shell or inner liner to air dry only.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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