

Foot Impression Boxes



Phenolic foam boxes used tool to produce an impression of the patient's foot.

Product Features & Benefits:

- Used in large quantities throughout the orthotic industry, these phenolic foam boxes are a quick and easy to use tool to give impression of the patient's foot.
- The foam is fine and soft to prevent skin irritation.
- Two blocks of phenolic foam impression material in a hinged corrugated board box.
- One block of impression material is held in each half of the box.
- Foam compresses under load and retains a dimensionally stable impression of the foot when load is removed.
- Sold per Pack of 25 Boxes.
- CE Mark

Instructions for use:

- Patient places foot on foam and presses down, in a neutral or weight bearing position as required.
- Negative mould is then filled with Plaster of Paris for positive cast and rectified to suit.

Cautions:

- The patient should not have the foot wet before placing it in the box.

Warranty:

- 12-month cover for defects in materials and workmanship in products purchased from Algeos only.

End of Use of the product:

Foam materials may be recycled. Follow local environmental guidelines for waste disposal.



Code	Type	Foam Depth	Foam Length
SA0451	Standard	40 mm	320 mm
SA0451	Deep	55 mm	320 mm