

# Bodytonix Post Op Shoes

Bodytonix



This all Purpose/Post Op Temporary Shoe is designed for use after post-operative procedures within wound care and diabetic treatment, or to protect the foot after trauma. Made with a soft nylon mesh upper and a semi-rigid sole to provide both comfort and stability to the patient while they recover.

## Indications for use:

- Post-surgery
- Diabetic wound care and oedema
- Trauma

## Product Features & Benefits:

- Semi-rigid sole provides ultimate protection and control while the reinforced heel counter provides extra stability for the rearfoot.
- Provides the foot with increased protection and accommodates bandages and cast
- Circumferential fabric instep cover to accommodate an oedematous foot
- Soft nylon mesh cover is breathable providing comfort for the wearer.
- Hook and loop strapping and dual side closure system to accommodate bulky dressings allow for a customised and adjustable fit throughout aftercare.
- Padded heel counter for increased comfort.
- Breathable open-toe design, closed heel.
- Anti-slip sole. Cushioned inlay.
- Universal left or right.
- CE Mark, Class I medical device.



## Instructions for use:

- Open all the hook and loop straps.
- Slowly slip the foot into place like normal shoes, taking care of injured areas.
- Fasten the strap closest to the toes then move up to the ankle.
- Tighten the straps so they are snug but not too tight. The shoe should not cut off the blood flow.
- Check the patient is comfortable and the straps are firmly in place.

## Cautions:

- Do not drive wearing this product.
- The use of the brace should be supervised by a physician or health care specialist. In case of any abnormal sensation such as tingling, burning or numbness, pain, swelling or skin irritation, discontinue use and seek medical assistance.

## Warranty:

- 12-month cover for defects in materials and workmanship in products purchased from Algeos only.

## Care and Maintenance:

- Wash it by hand in cold water with a mild detergent. Do not use a washing machine or dryer. Can also be cleaned with a damp cloth and mild soap.
- Reusable if not used on open wounds.

## End of use of the Product:

- Metal and Plastic materials may be recycled. Follow local environmental guidelines for waste disposal.

Code	Size	UK Shoe size
SP1012	Small	3 - 5
SP1013	Medium	6 - 7
SP1014	Large	8 - 9
SP1017	X Large	10 - 12



A. Algeo Ltd  
Sheridan House  
Bridge Industrial Estate  
Speke Hall Road  
Liverpool  
L24 9HB, UK

+44 (0)151 448 1228  
+44 (0)151 448 1008  
algeos.com  
info@algeos.com