

Bodytonix Adjustable Night Splint

Technical characteristics and usage advice

The Bodytonix Adjustable Night Splint is a universal left or right fitting splint that can be used to treat multiple causes of heel pain and support correction.

Features & Benefits:

- Designed to comfortably stretch the plantar fascia and Achilles tendon during sleep.
- Helps reduce muscle contracture, inflammation and pain associated with the first step in the morning or after periods of rest.
- Adjustable flexion straps provide gentle stretching of the plantar fascia and Achilles tendon.
- Lycra® lined foam interfaces provide maximum comfort and eliminate heat build-up.
- Durable and cost-effective.
- Flexible and lightweight.
- Breathable, hygienic and washable components.
- Suitable for left or right foot.
- Foam toe wedge included for increased plantar and dorsiflexion
- Available in Small, Medium and Large.

Technical Characteristics:

- Adjustable dorsiflexion from 90° to 10°.
- Adjustable flexion straps.
- Low profile shell.
- Foam toe wedge.
- Removable liner.
- Soft Lycra® lined foam interior.
- Lycra® lined foam padded straps.
- Soft pads: Polyester.
- Velcro® straps: Nylon.
- Outer shell: Plastic.
- Stretching wedge: EVA with Velcro® backing.

Indications for Use:

Plantar Fasciitis, Calcaneal Spur, Pronation, Achilles Tendonitis and Plantar Flexion Contractures.

Cautions:

- Its use should be supervised by a physician or health care specialist, who should provide direction on frequency and duration of use.
- In case of any abnormal sensation such as tingling, burning or numbness, pain, swelling or skin irritation, discontinue use and seek medical assistance.



Code	Size (UK Shoe Size)
SW1202	Small (Up to Size 5)
SW1203	Medium (6-8)
SW1204	Large (9-12)

Storage:

Store at room temperature. Do not expose to direct sunlight.

Markings:

This product contains no additional markings other than the brand label.

Care Instructions:

Can be wiped clean with a damp cloth using a mild soap or disinfectant. Do not dry on a radiator.

Warranty:

12-month cover from purchase for defects in materials and workmanship.