

Malleo Ankle Support

This ankle bracelet provides support for damaged tissues and prevents further injury. The compression effect ensures a high degree of support of the ligaments and relieves the joint pressure, reducing pain and swelling. It is suitable for sprains and strains as well as arthritic and ligament pain or other inversion/ eversion injuries.

Indications for use:

- Post-surgical, post-plaster cast or post-traumatic recovery and immobilisation
- Mild ligament injury and instability
- Faster reabsorption of edemas, effusions and hematomas
- Irritations of the talocrural joint
- Osteoarthritis and arthritis
- Early mobilisation

Product Features & Benefits:

- Manufactured anatomically using the unique 'Auractive' technology, where neoprene material is laminated between two layers of jersey material, providing ease of movement and soothing effect as well as retaining high strength that conforms intimately to the ankle, heel and malleoli.
- This technology also promotes the healing process by increasing blood circulation.
- Semi rigid plastic shells lateral to ankle joint support the inner and outer malleolus.
- Elastic wrapping around the ankle with the shape of an 'eight' relieves the edema which reduces the pain.
- Pull-on design with hook and loop strapping system to enhance inversion control at sub-talar joint and reduce malleolar drift.
- Universal left or right. Can be used with shoes.
- CE Mark, Class I medical device.

Instructions for use:

- Open the velcro strips and put on your foot like a sock. Close the velcro strips for a tight fit.
- It is highly recommended that a sock be worn when using this product.
- Ensure blood circulation supply is not restricted by selecting the right size.

Cautions:

- Do not drive while wearing this brace.
- The use of the brace should be supervised by a physician or health care specialist. In case of any abnormal sensation such as tingling, burning or numbness, pain, swelling or skin irritation, discontinue use and seek medical assistance.

Warranty:


- 12-month cover for defects in materials and workmanship in products purchased from Algeos only.

Care and Maintenance:


- Clean with a wet cloth and mild detergent. Also washable by hand with mild detergent in cold water. Air dry.
- Reusable if not used on open wounds.

End of use of the Product:

- Plastic-based materials may be recycled. Follow local environmental guidelines for waste disposal.

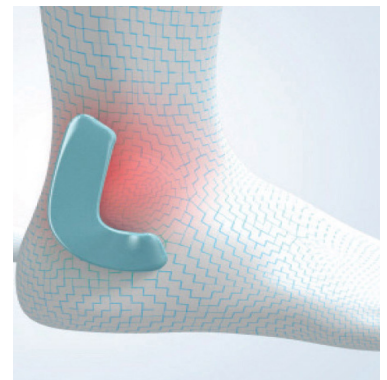

Auractive

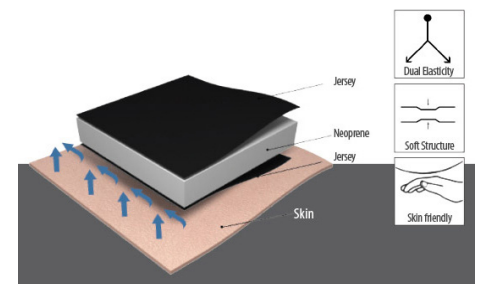

Code	Size	Measurements	Colour
SW2014	Small	21-23 cm	Red & Black
SW2016	Medium	23-25 cm	
SW2018	Large	25-27 cm	
SW2020	Extra Large	27-29 cm	
SW2022	Extra Extra Large	29-31 cm	



Distributed by A.Algeos Ltd, Sheridan House, Bridge Industrial Estate, Speke Hall Rd, Liverpool L24 9HB, UK.
Contact: info@algeos.com

Manufactured by Aurafix Health Products. Adil Mah. Ekol
Cad. No:12/A 34935 Sultanbeyli / Istanbul