HeelWedge®

EN Off-loading Shoe
DE Fersentlastungsschuh
ES Zapato de descarga del talón
FR Chaussure de décharge du talon
IT Calzatura per lo sgravio del tallone
NL Hielhulpschoen
DA Hælaflastningssko
SV Hälavlastningssko
PL But odciążający ścięgno
SL Čevlji za razbremenitev pete
HR Čipela za rasterećenje pete
PT Sapato de alívio do calcanhar

DARCO

HeelWedge® | Off-loading Shoe

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility for misuse of the product does not assume any responsibility. Do not wear the product with injured, irritated, damaged or broken skin. You should use bandages or socks, for example.

Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal.

This device is intended for single-patient use only.

Purpose

The HeelWedge® off-loading shoe is designed to promote healing by reducing weight from the rearfoot. It and may be used for any condition requiring weight or pressure reduction from the metatarsal heads proximally for conditions such as ulcers, infections, trauma and surgery.

Indications / Contraindications

Indications

- > Postoperative healing for either soft tissue or bony structure of the heel
- > Selective off-loading after fractures
- > For the treatment of ulcerations present on the heel area (rear foot ulceration)

Contraindications

- > Open fractures

Design features

The shortened sole provides effective pressure off-loading. The removable insole can be exchanged by the PegAssist® Insole.

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the duration of use prescribed by a doctor (usually about 6 weeks) without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your medical professional.

EN Use

1. Open the shoe (A).
2. Insert the foot into the opened shoe.
3. Make sure that the foot is positioned securely in the shoe.
4. Close the shoe with the hook and loop fasteners (A). So that the fit is snug, but not too tight.
5. Only roll as far as the inner edges (B) when walking. Rolling as far as the outermost edges (C) can have a negative effect on the results of surgery.
6. When worn for longer periods, it is recommended that the height of the other foot is adapted to match.

Technical data

Sizes	XS (33.5 – 37), S (37.5 – 39.5), M (40 – 41.5), L (42 – 44), XL (44.5 – 47) Universal fit for right and left
Colour	Black
Material	Outsole: TPR Insole: EVA Upper material: Polyester Hook and loop fastener: Nylon

Combination with other products

Recommended add-ons

- > TwinShoe® - Balance shoe,
- > PegAssist® Insole - Individually customizable insole for targeted off-loading

Maintenance

The product does not require maintenance.

Cleaning instructions

Hand wash

- > When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.
- > The shoe may not tumbled dry or dried on the heater. Allow the Schuh to air dry at moderate temperatures.

Disposal

Please dispose of the packaging in an environmentally friendly manner. When treatment is complete, dispose of this device according to local laws and ordinances.

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DARCO International, Inc.
V-M Orthotics Ltd.
ALBO-Healthcare GmbH