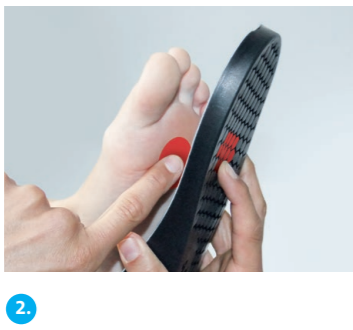
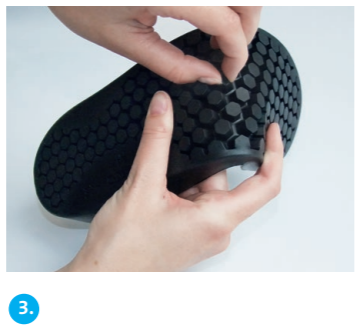
1.

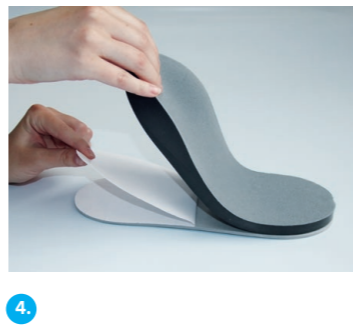
2.


3.

4.

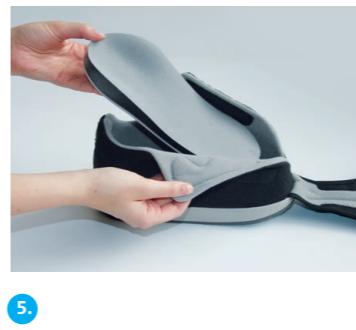5.

**USA/GB Instructions for use**

**PegContour® Insole**  
Off-loading Cushion Insole

The PegContour® Insole is an effective, three-dimensional insole system for redistribution of plantar pressure. Removal of single pegs enables customization – postoperatively or for acute wound care. A thin layer made of EVA (sole base) stabilises the off-loading area, and keeps the remaining pegs in place. The lateral and medial longitudinal arch and the proximal transverse arch provide optimal pressure redistribution. The pressure is reduced by up to 55%.

**Recommendations for use**

1. Remove the grey or black original sole from the shoe (fig. 1).
2. Determine the area of the foot that needs relief (fig. 2).
3. Carefully remove the pegs at the site at which the pressure should be relieved (fig. 3). Use a suitable tool for this or remove the pegs by hand.
4. Remove the anti-adhesive paper and carefully affix the complete surface of the sole base (supplied) to the pegs. Thus, the pegs are fixed around the pressure relief zone. This ensures that the area needing relief does not shift (fig. 4).
5. Reinsert the adapted PegContour® Sole in the shoe with the affixed sole base facing downwards (fig. 5).
6. Fitting finished.

**Indication**

- > for use after hallux valgus and small toe surgeries (Austin, Akin, Scarf, Weil, Hohmann, etc.)
- > talipes valgus, flat foot, splayfoot
- > angioneuropathic changes of the foot in combination with mild foot deformities

**Additional indications for the blue sole**

- For selective off-loading for:
- > plantar lesions
- > plantar ulcers / ulcer prevention
- > pressure points

**Additional indications for the black sole**

- > healing of ulcers in the heel area (rearfoot ulcers)
- > heel spurs (plantar fasciitis)
- > heel wounds
- > postoperative use and treatment of the heel

**Contraindication**

- > open fractures
- > marked foot deformities of the diabetic foot

**Cleaning instructions**

The sole can be cleaned with water at approx. 30° C or lukewarm water or with conventional disinfectants. Do not dry the sole on a heating unit; instead allow it to dry at room temperature.

**Technical data**

- Size:**
- > available in six or seven sizes
- > right and left version available

The insole is available in the corresponding sizes for the following aids with the pegs in one of two colours (blue or black – according to the model):

- > AllRound Shoe® – Closed Wound Care Shoe
- > Relief Dual® – Off-loading Shoe
- > MedSurg, MedSurg Pro® – Healing Shoe

**Material:**

The insole with a height of 15 mm is made of polyurethane. The sole covering is made of polyester. The separate sole base is made of EVA (40° Shore).

**Maintenance:**

The product does not require maintenance.

**Construction features:**

The PegContour® Insole, made 100% from PU material, provides perfect pressure redistribution with its transverse arch as well as medial and lateral longitudinal arch. The lengthwise and crosswise sole covering provides a high level of wearing comfort. Easy removal of single pegs enables selective work. A thin sole stabilises the off-loading area, and keeps the remaining pegs in place.

**Caution**

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. The function and optimal protection are no longer guaranteed if the product is incorrectly used. Signs of fatigue can occur in the cover layer when there is a large gap. This appliance, together with a special shoe, is intended to assist you in your recovery and should only be used as prescribed by (or in consultation with) your doctor or medical specialist. DARCO (Europe) GmbH does not assume any responsibility. It is recommended that dressings or socks are worn. We recommend that you do not drive a vehicle while you are wearing this aid in a special shoe. Take extreme caution and go more slowly than usual. This aid is intended for single-patient use only.

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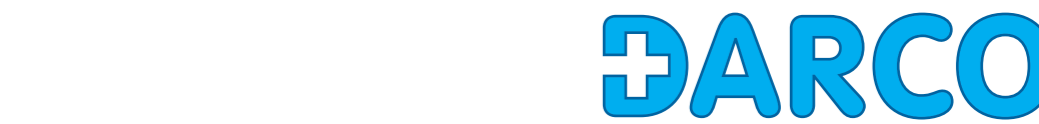
USA/GB  
For pressure redistribution  
and selective off-loading

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**Indication**

- > for use after hallux valgus and small toe sur-ges (Austin, Akin, Scarf, Weil, Hohmann, etc.)
- > talipes valgus, flat foot, splayfoot
- > angioneuropathic changes of the foot in combination with mild foot deformities

**Indications for the blue sole**

- > plantar lesions
- > plantar ulcers / ulcer prevention
- > pressure points

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**Features**

- > effective insole system for plantar pressure relief in acute care
- > pressure can be reduced by up to 55%
- > optimal pressure redistribution is achieved through the contours
- > soft lengthwise and crosswise sole covering made of functional polyester
- > customizable PU material
- > customised fit through the removal of single pegs
- > available for the right and left foot

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