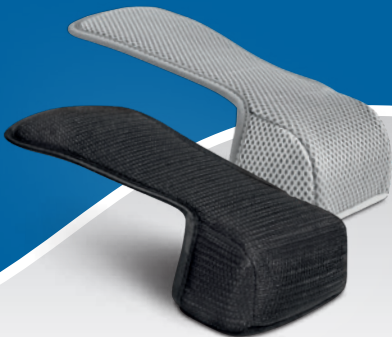
Toe Cap

EN Toe Cap
DE Zehenkappe
ES Puntera
FR Protège-orteils
IT Mascherina salvadita
NL Teenkap
DA Tåkappe
SV Tåkappa
PL Osłona na palce nóg
SL Kapica za nožne prste
HR Kapica za nožne prste
PT Proteção do dedo grande do pé

DARCO

Toe Cap

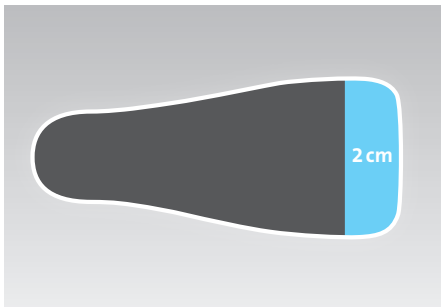
1

2

3

4

EN Instructions for use

Toe Cap | Toe Cap

Important notes

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly.

DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example.

Purpose

The Toe Cap (photo 1) allows the DARCO orthopaedic aid to be worn in closed form. It protects the foot from cold and dirt and improves comfortable fitting.

Technical data

- Sizes** Available in all corresponding shoe sizes for the products:
Universal fit for right and left: OrthoWedge, TwinShoe®, MedSurg/Pro®, Relief Dual®/Plus, Relief Insert® Walker 2.0/Achill
Right and left version available: WCS®
- Colours** Black (WCS®, OrthoWedge, TwinShoe®, MedSurg/Pro®)
Grey (Relief Dual®/Plus, Relief Insert® Walker 2.0/Achill)
- Materials** WCS®: Elastane / polyester
OrthoWedge, TwinShoe®, MedSurg: Dacron® polyester
MedSurg Pro®: Foam polyester mesh
Relief Dual®/Plus, Relief Insert® Walker 2.0/Achill: Polyester foam, microfibre

EN Use

1. Take out the removable insole of the orthopaedic aid.
2. Remove the hook and loop fastener from the Toe Cap and now remove the white adhesive strip. (photo 2)
3. Then affix the hook and loop fastener parallel to the front edge of the sole onto the inner side of the outsole. Please note that the hook and loop fastener should not be affixed more than a maximum of 2 cm from the front edge of the sole. (photo 3)
4. Now slide the insole together with the Toe Cap into the shoe.
5. Press the insole gently with your hand so that it is fixed securely and close the shoe as usual. (photo 4)

Working life / Lifespan of the product

The lifespan of the product is determined by natural wear and tear due to proper handling / cleaning.

Maintenance

The product does not require maintenance.

Cleaning instructions

The DARCO Toe Cap must be washed by hand.

Disposal

 Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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A. Algeo Ltd